



Rhino
Rescue

COMPRESSED GAUZE

-ALWAYS BE PREPARED-

COMPRESSED GAUZE

SPECIFICATIONS

BANDAGE: 4.5in × 4.1yds
(115mm × 3.75m)

WEIGHT: 40g

MATERIAL: Cotton


ITEM#

PZSB0034


DESCRIPTION

RHINO RESCUE compressed gauze is made from 100 % premium cotton for superior absorbency, stability, and softness, while remaining breathable, flexible, and tear-resistant. Z-folded and vacuum-sealed into an ultra-compact package, it slips easily into any IFAK or medic bag. Use it with an emergency bandage for rapid hemorrhage control, as backing for hemostatic dressings, or for minor wound care—an indispensable component of tactical first aid.


INSTRUCTIONS FOR USE

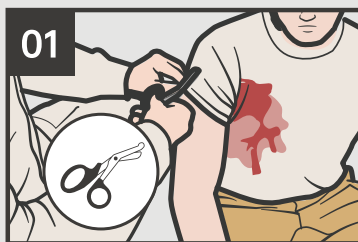

CUT the clothes of the injured part with scissors.

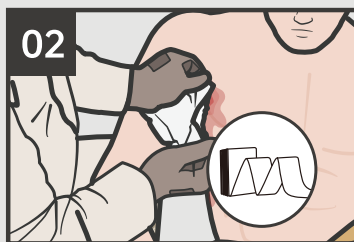

PREPARE compressed gauze to fill the wound.

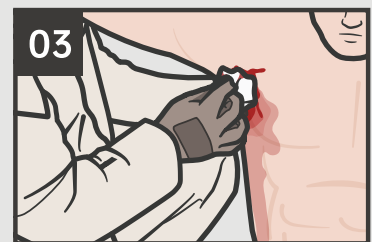


EXTEND the casualty's arm at a 90-degree angle by placing it on your shoulder.



PACK the wound tightly with compressed gauze until the wound cavity is filled.



ENSURE the gauze extends 1-2 in above the skin.

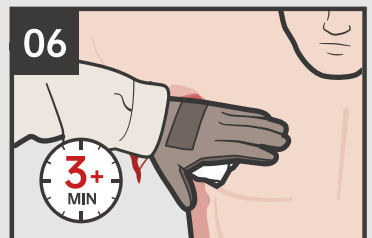

HOLD pressure for a minimum of 3 min to ensure that the wound does not bleed.

MD CE 0197 STERILE R !

2 3 4 5 6 7 8 9 10 11 12

EU REP SUNGO Cert GmbH, Harlfstr.47, 40591 Düsseldorf, Germany E-mail: de.rep@sungoglobal.com



EMSRUN GROUP
Room 101,201,301,No.9 Building,Huagang Creative
Park,Huishan Area,Wuxi,214000 Jiangsu,P.R.China
E-mail: info@rhinorestore.com



WUXI EMSRUN TECHNOLOGY CO., LTD.
9-201, Huagang Chuangzhi Park, Huishan Economic
Development Zone, Wuxi, Jiangsu Province, CHINA
E-mail: info@emsrn.com

www.rhinorestore.com | info@rhinorestore.com