

ELASTIC CREPE BANDAGE

SPECIFICATIONS

SIZE: 3in × 5yds (75mm × 4.5m)

MATERIAL: Cotton

ITEM#

CPBD0016




DESCRIPTION

The Rhino Rescue Elastic Crepe Bandage is an essential part of any first aid or trauma kit as it can be used to immobilise sprains and fractures and can be used for other applications as well, such as holding trauma gauze in place or providing external pressure to wounds.


INSTRUCTIONS FOR USE




WRAP the elastic bandage around injured shoulder twice ensuring the gauze underneath is completely covered.

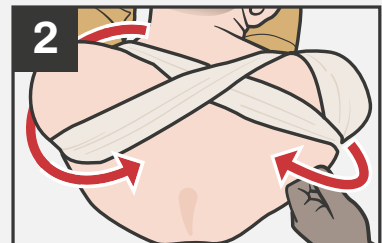

WRAP elastic bandage across, back and under the opposite armpit, anchoring around the opposite shoulder in a "figure 8" pattern.

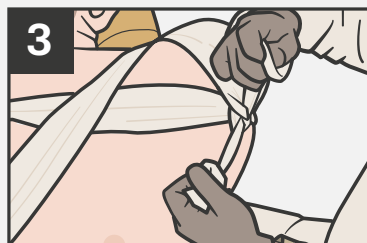

After bandaging, **SECURE** with tie tails of elastic bandage.

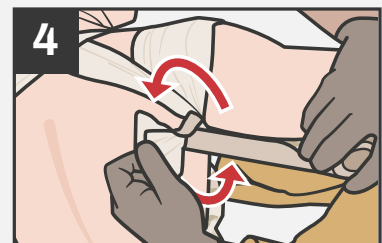

SECURE elastic bandage tails and knot using tape, prevent the knots from unraveling.



EU REP SUNGO Cert GmbH,
Harfstr. 47, 40591 Düsseldorf, Germany
E-mail: de.rep@sungoglobal.com



EMSRUN GROUP
8-7 Huide Road, Huishan Area, Wuxi, 214000
Jiangsu, CHINA
E-mail: info@rhinorestore.com



WUXI EMSRUN TECHNOLOGY CO., LTD.
Room 501, No. 8-7 Huide Road, Wuxi City, Jiangsu
Province, China
E-mail: info@emsr.com

www.rhinorestore.com | info@rhinorestore.com