



Rhino
Rescue

EMERGENCY BANDAGE

-ALWAYS BE PREPARED-

EMERGENCY BANDAGE

DESCRIPTION

RHINO RESCUE pressure trauma bandage is a tactical-grade hemorrhage-control device. Its patented hemispherical dome generates focused pressure to accelerate clotting. Integrating a sterile, non-adherent cotton pad, built-in pressure bar, elastic wrap, and fool-proof securing system, it enables both professionals and civilians to achieve rapid self-application. Military-tested, it delivers precise compression, doubles as an arm sling or splint immobilizer, and handles any task requiring elastic support—yet remains compact and rugged enough for virtually any IFAK.




Regular Bandage




**NEW
UPGRADE!**

True Pressure Bandage

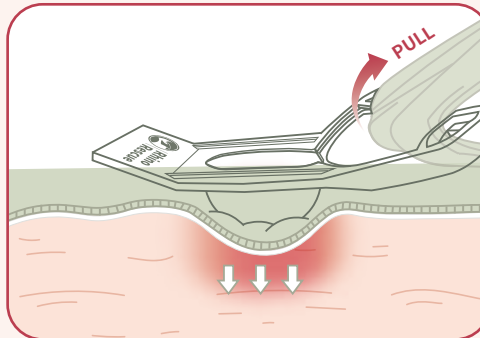


- 1** Increasing the pressure on the wound
- 2** Without changing The original Operating method
- 3** Accelerate haemostasis


INSTRUCTIONS FOR USE

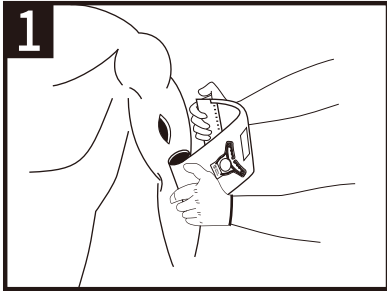


1 PLACE the pad directly on the wound and maintain pressure.

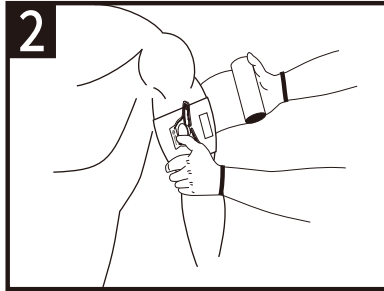


2 WRAP the elastic bandage around injured extremity and maintain pressure.

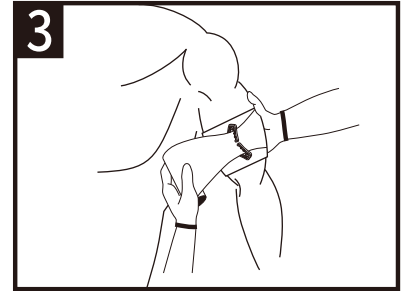


3 INSERT the elastic bandage into the pressure bar.

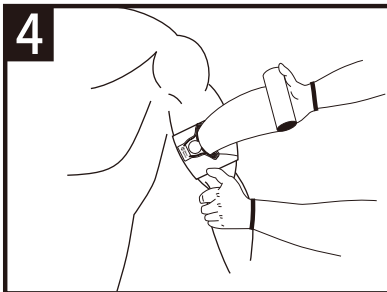

4 REVERSE WRAP the elastic bandage back over the top of the pressure bar.

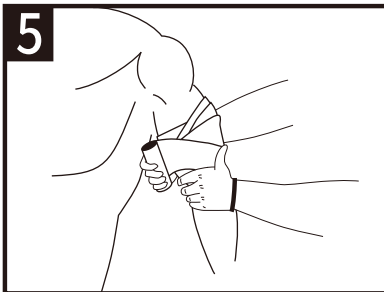

5 WRAP bandage tightly over the pressure bar until pad edges are covered.

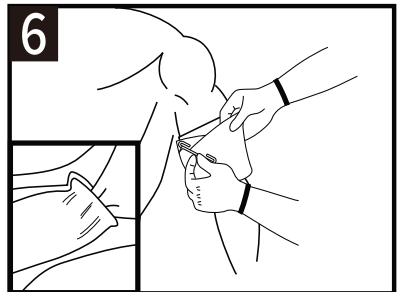

6 SECURE the hooking end of the pressure bar into the elastic bandage.

DETAILED FEATURES **EMERGENCY BANDAGE TRUE PRESSURE BANDAGE**




Anti-Drop
Prevent the bandage from unraveling.



Elastic Wrap
Apply enhanced pressure to bring about hemostasis.

Sterile Dressing
Keep the wound clean and pressure the wound firmly.

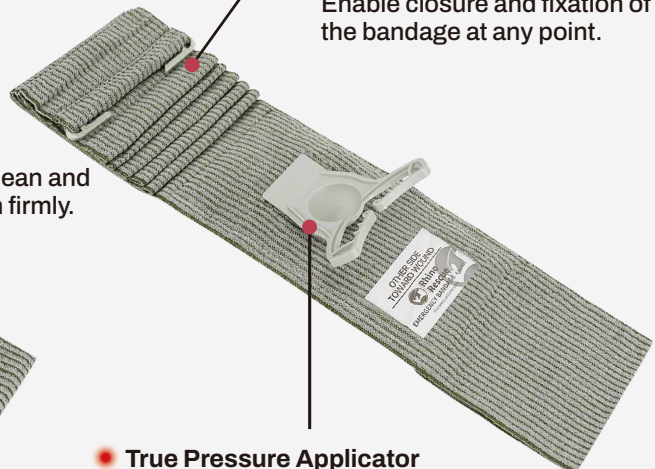

Closure Bar
Enable closure and fixation of the bandage at any point.

True Pressure Applicator
Hemispherical raised structure increases the pressure on the wound.




DETAILED FEATURES

TRAUMA BANDAGE



DETAILED FEATURES

EMERGENCY BANDAGE SKIN TONE BANDAGE

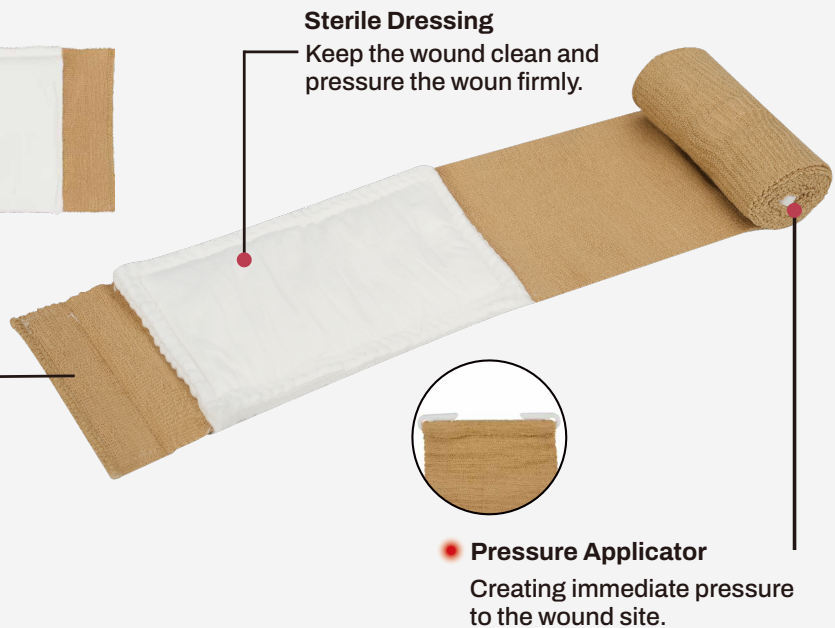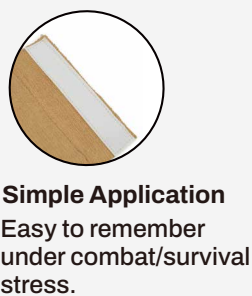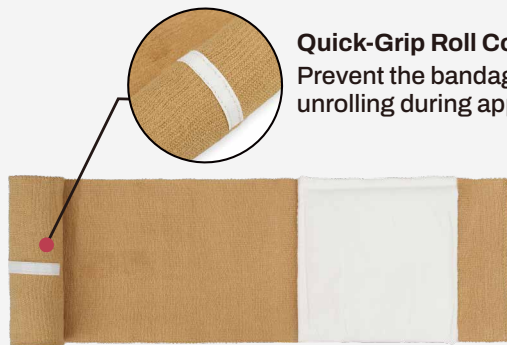

SPECIFICATION



Emergency Bandage True Pressure Bandage

SIZE	ITEM #	Weight
4in (100 × 1300mm)	PZBD0161	70g
6in (150 × 1300mm)	PZBD0162	95g



Emergency Bandage

SIZE	ITEM #	Weight
4in (100 × 1300mm)	PZBD0142	63g
6in (150 × 1300mm)	PZBD0143	90g



Emergency Bandage Skin Tone Bandage

SIZE	ITEM #	Weight
4in (100 × 1500mm)	PZBD0144	63g
6in (150 × 1500mm)	PZBD0145	110g
8in (200 × 1500mm)	PZBD0146	170g



SUNGO Cert GmbH.
Harffstr.47,40591 Düsseldorf, Germany
E-mail: de.rep@sungoglobal.com



EMS RUN GROUP
Room 101,201,301,No.9 Building,Huaqing Creative
Park,Huishan Area,Wuxi,214000 Jiangsu,P.R.China
E-mail: info@rhinorescuestore.com



WUXI EMSRUN TECHNOLOGY CO., LTD.
9-201, Huaqing Chuangzhi Park, Huishan Economic
Development Zone, Wuxi, Jiangsu Province,CHINA
E-mail: info@emsrun.com