

EMERGENCY LITTER



DESCRIPTION

RHINO RESCUE EMERGENCY LITTER is a highly efficient transport tool specifically designed for emergency medical rescue. Its foldable design allows for easy carrying and storage, taking up minimal space and making it suitable for immediate emergency use. Whether in an ambulance, battlefield rescue, outdoor adventure, or sports event, the soft emergency stretcher is the ideal choice for quick and safe patient transport.

• PRODUCT UPGRADE

• OLD

• NEW



INTEGRATED USE AND STORAGE

FEATURE

- Foldable for storage, space saving and easy to carry.
- Features ten large carry handles with a load capacity tested to 300kg.
- Water and tear resistant, good load-bearing capacity.
- Comes with a storage bag for easy protection of the product.

SPECIFICATION

PRODUCT SIZE: 186×68cm

WEIGHT: 365g

PRODUCT MATERIAL: NYLON

PACKAGED SIZE: 17×25cm

LOAD RATING: 300kg

• STORAGE STEPS

STEP 1:

Spread it out



STEP 2:

Fold it horizontally



STEP 3

Fold it horizontally again



STEP 4

Roll it up vertically



STEP 5

Roll into the bag



GENERAL

- Do not use Emergency Litter if unit is excessively worn or punctured.
- Avoid contact with sharp objects.
- Avoid dragging Multi-Use Disposable Litter over rough surfaces.
- Do not store Emergency Litter in contact with heat sources greater than 190°F.
- Not to be used in conjunction with any type of mechanical lifts for either vertical or horizontal movement.